




# Datasheet collection




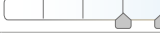



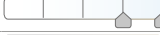
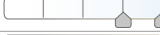




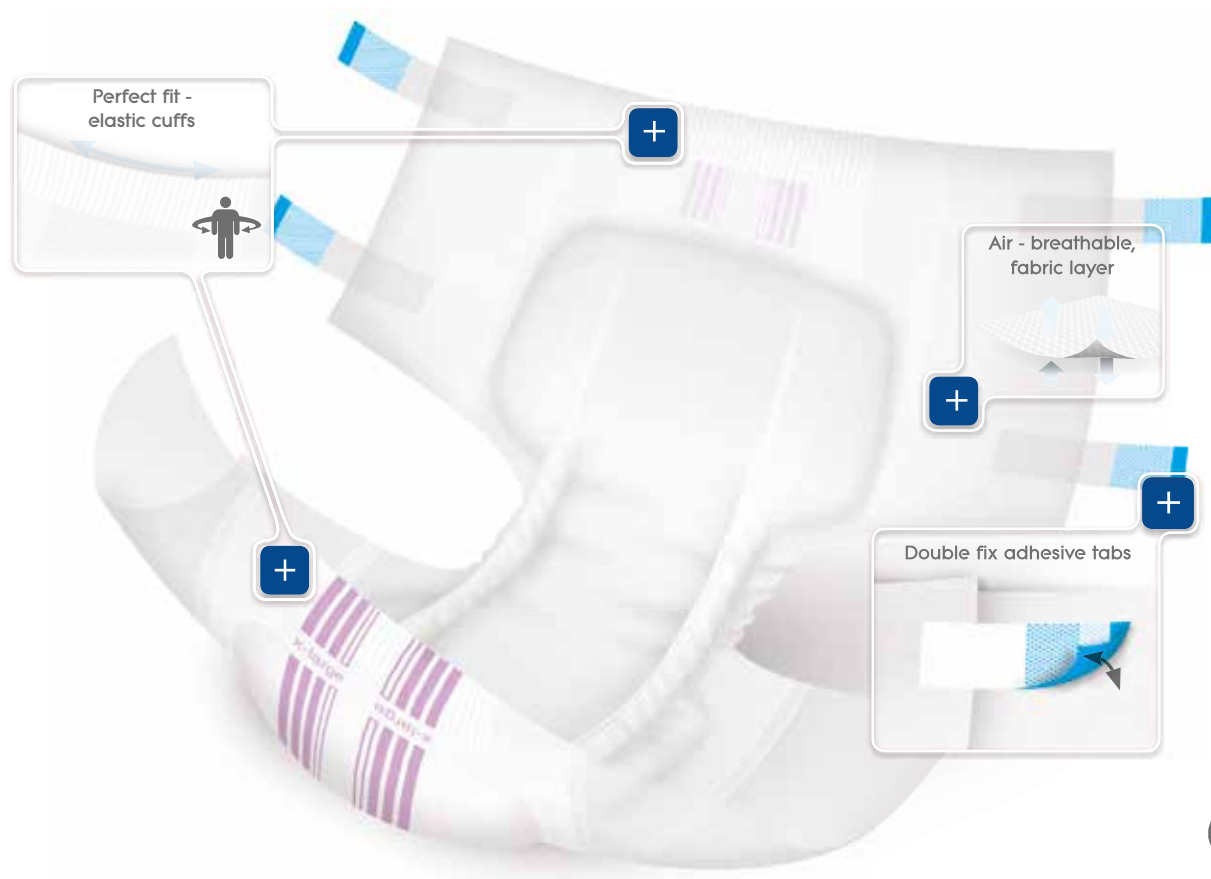








## Incontinence severity (involuntary loss of urine in 4 hours)



## The best thing we have ever developed!

forma-care slip is a very absorbent, anatomically shaped incontinence shaped pad with breathable air TBS outer foil with felt in the hip area. The dermatologically tested and latex free product offers maximum comfort.




	NPC	MPC	EAN code	Brand name	code	Severity	Waist size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
Sensitive	CFQ947	5155	4036978900291	forma-care slip Sensitive - plus s			50 - 90 cm	10	20	2100
	CFQ948	5065	4036978900307	forma-care slip Sensitive - super medium			70 - 125 cm	9	20	2480
	CFQ952	5562-01	4036978900314	forma-care slip Sensitive - extra medium			70 - 125 cm	11,1	18	3200
	CFP1987	5563-01	4036978013649	forma-care slip Sensitive - x plus medium			70 - 125 cm	10	18	3330
	CFQ949	5070-01	4036978900321	forma-care slip Sensitive - super large			100 - 155 cm	10,9	20	2500
	CFQ950	5567-01	4036978900338	forma-care slip Sensitive - extra large			100 - 155 cm	11,9	16	3800
	CFQ953	5567	4036978900338	forma-care slip Sensitive - extra large			100 - 155 cm	11,9	16	3950
	CFQ955	5568	4036978013663	forma-care slip Sensitive - extra plus large			100 - 155 cm	8,9	16	3950
	CFQ951	1124	4036978004791	forma-care slip PREMIUM - bariatric			160 - 190 cm	8,4	16	2980
	CFQ954	1124-01	4036978004791	forma-care slip PREMIUM - x large			115 - 175 cm	8,4	16	2980
Comfort	CFQ956	4156	4036978900369	forma-care slip comfort - small plus			50 - 90 cm	12	20	2790
	CFQ957	6065	4036978900178	forma-care slip comfort - medium plus			70 - 125 cm	8,9	20	2588
	CFQ958	9560	4036978900161	forma-care slip comfort - medium super			70 - 125 cm	10	20	3178
	CFQ961	9562	4036978900109	forma-care slip comfort - medium extra night			70 - 125 cm	13,9	18	3250
	CFP1989	9563-01	4036978900147	forma-care slip comfort - x plus medium			70 - 125 cm	11,5	18	4467
	CFQ959	6070	4036978900185	forma-care slip comfort - large plus			100 - 155 cm	10,9	20	3280
	CFP1986	6070-01	4036978900185	forma-care slip comfort - plus x large			115 - 175 cm	10,9	20	3280
	CFQ962	9565	4036978900192	forma-care slip comfort - large super			100 - 155 cm	14,9	20	4000
	CFQ965	9568	4036978900154	forma-care slip comfort - large x plus			100 - 155 cm	11,5	16	4790
	CFP1988	9568-01	4036978900154	forma-care slip comfort - x plus large			100 - 155 cm	11,5	16	4790
	CFQ960	4169	4036978900345	forma-care slip comfort - super extra large			115 - 175 cm	8,4	16	3890
	CFQ963	4169-01	4036978900345	forma-care slip comfort - super extra large			115 - 175 cm	8,4	16	3890
	CFQ964	2124	4036978900352	forma-care slip comfort - extra x large			115 - 175 cm	11	16	4790



## How to fit whilst standing



Unfold the All-in-One and stretch out into the shape of a "ship". At the back of the product you will find the adhesive strips. Place the All-in-One from between your legs in the shape of a "ship" pulling through from front to back.

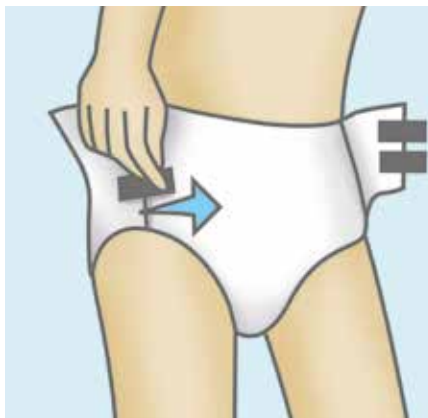


Smooth out the front side of the product. Unfold the back of the product - Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.




When applying the front, fold the All-in-One sideways, do not pull up the product upwards. The lower adhesive strips fix horizontally, the upper adhesive strips, downwards and diagonally.

## How to fit whilst lying down

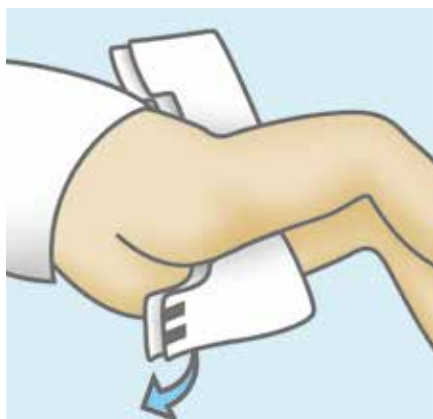

Unfold the All-in-One and stretch out into the shape of a "ship". At the back of the product you will find the adhesive strips. Place the All-in-One from between your legs in the shape of a "ship" pulling through from front to back.

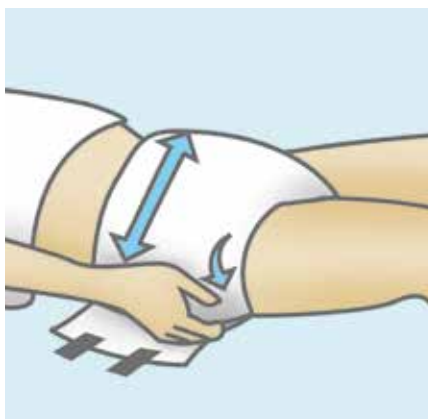

Smooth out the front side of the product. Unfold the back of the product - Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.



When applying the front, fold the All-in-One sideways, do not pull up the product upwards. The lower adhesive strips fix horizontally, the upper adhesive strips, downwards and diagonally.

### User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.

## Fixation pants

Fixation pants are the perfect addition to shaped pads. The material structure enables an optimal microclimate and the material composition offers great elasticity. They are a great allrounder, the fit is close to the body and support the person's mobility.

The fixation pants have a particularly soft leg cuff made out of microfiber and higher waist band to ensure an improved fixation even of larger pads. Washable up to 60 ° C. Not suitable for Machine Drying. Fixation pants are the perfect addition to shaped pads.

CE



# Datasheet

## Fixation pants



PREMIUM	NPC	MPC	EAN code	Brand name	Colour code	Waist size	Weight in kg	Contents/pack
	CFP1994	52-01-210	4036978022931	forma-care net pants for use with pads / with legs / Small (Seamless BASIC)		40 - 80 cm	0,9	100
	CFP1995	51-01-230	4036978022917	forma-care net pants for use with pads / with legs / Small (COMFORT LINE)		50 - 90 cm	0,9	100
	CFP1996	51-02-330	4036978022924	forma-care net pants for use pads / with legs / Medium (COMFORT LINE)		80 - 120 cm	0,9	100
	CFP1997	10-01-210	4036978022825	forma-care net pants for use with pads / without legs / Small (BASIC LINE disposable)		50 - 90 cm	0,9	100
	CFP1998	10-02-710	4036978022832	forma-care net pants for use with pads / without legs / Medium (BASIC LINE disposable)		80 - 120 cm	0,9	100
	CFP1999	10-03-000	4036978022849	forma-care net pants for use with pads / without legs / Large (BASIC LINE disposable)		100 - 140 cm	0,9	100
	CFP2000	10-04-310	4036978022856	forma-care net pants for use with pads / without legs / X Large (BASIC LINE disposable)		120 - 160 cm	0,9	100
	CFP2001	10-05-130	4036978022863	forma-care net pants for use with pads / without legs / XX Large (BASIC LINE disposable)		140 - 180 cm	0,9	100
	CFP2002	20-02-310	4036978022870	forma-care net pants for use with pads / without legs / Medium (STANDARD)		80 - 120 cm	0,9	100
	CFP2003	20-03-610	4036978022887	forma-care net pants for use with pads / without legs / Large (STANDARD)		100 - 140 cm	0,9	100
	CFP2004	20-04-710	4036978022894	forma-care net pants for use with pads / without legs / X Large (STANDARD)		120 - 160 cm	0,9	100
	CFP2005	20-05-110	4036978022900	forma-care net pants for use with pads / without legs / XX Large (STANDARD)		140 - 180 cm	0,9	100



## Incontinence severity

(involuntary loss of urine in 4 hours)



## PREMIUM Dry flex

forma-care PREMIUM Dry flex is a highly absorbent and anatomically shaped incontinence with an air-permeable TBS outer film and an easy Flex belted fixation system around the waist.



CE





# Belted slip - flex




PREMIUM

NPC	MPC	EAN code	Brand name	Colour code	Severity	Waist size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
CFP1973	36007	4036978022573	forma-care flex PREMIUM - small plus			50 - 85 cm	6	25	1600
CFP1974	36008	4036978022597	forma-care flex PREMIUM - small super			50 - 85 cm	7,9	25	2000
CFP1980	36009	4036978022610	forma-care flex PREMIUM - small maxi			50 - 85 cm	7,6	22	2400
CFP1975	37001	4036978001561	forma-care flex PREMIUM - medium plus			70 - 115 cm	7,5	25	1900
CFP1977	37002	4036978022658	forma-care flex PREMIUM - medium super			70 - 115 cm	8,2	22	2400
CFP1981	37003	4036978022672	forma-care flex PREMIUM - medium maxi			70 - 115 cm	8,1	18	3000
CFP1976	37004	4036978001622	forma-care flex PREMIUM - large plus			85 - 135 cm	7,9	25	2000
CFP1978	37005	4036978022719	forma-care flex PREMIUM - large super			85 - 135 cm	7,6	18	2600
CFP1982	37006	4036978022733	forma-care flex PREMIUM - large maxi			85 - 135 cm	7,7	16	3500
CFP1979	37007	4036978022757	forma-care flex PREMIUM - x-large plus			100 - 155 cm	7,2	20	2400
CFP1983	37007-01	4036978022757	forma-care flex PREMIUM - extra large plus			100 - 155 cm	7,2	20	2400
CFP1984	37008	4036978022771	forma-care flex PREMIUM - x-large super			100 - 155 cm	7,7	16	3000
CFP1985	37009	4036978022795	forma-care flex PREMIUM - x-large maxi			100 - 155 cm	8,2	16	3800





## How to fit whilst standing

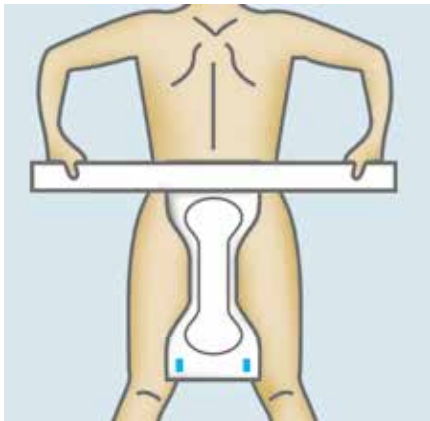

Fold the product along the center of the pad into the shape of a "ship", then peel off the protective film from the insert.




Place the insert with the adhesive side down into the underwear.




Pull up the underwear or net fixation pants to fit.


## How to fit whilst lying down

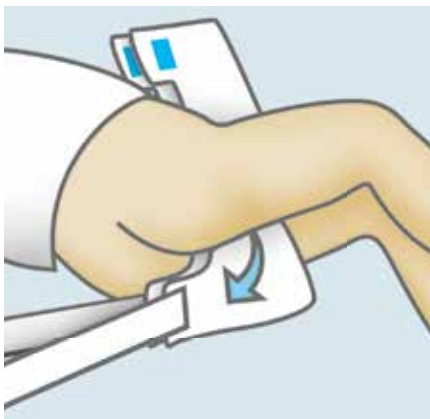

Remove the protective strip so that the adhesive surface is released.

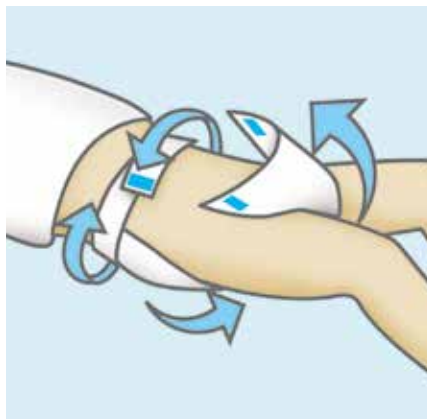

Fold the insert lengthwise to a small ship shape. Place the penis and testicles into the insert.

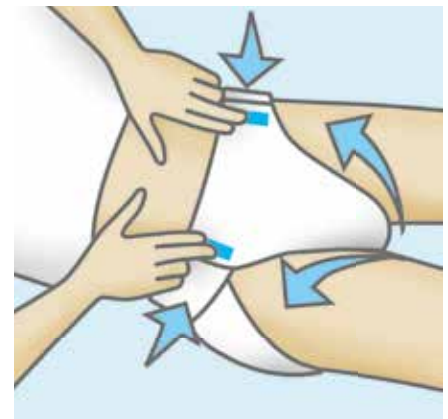

Pull up the underpants or fixation pants.

### User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.

# Large shaped pads




## Incontinence severity (involuntary loss of urine in 4 hours)



## Highest quality of life with absolute reliability.

forma-care form is an anatomically shaped incontinence pad with ventilated TBS-air foil backing. The dermatologically tested and latex free product excels through its super absorbency qualities.

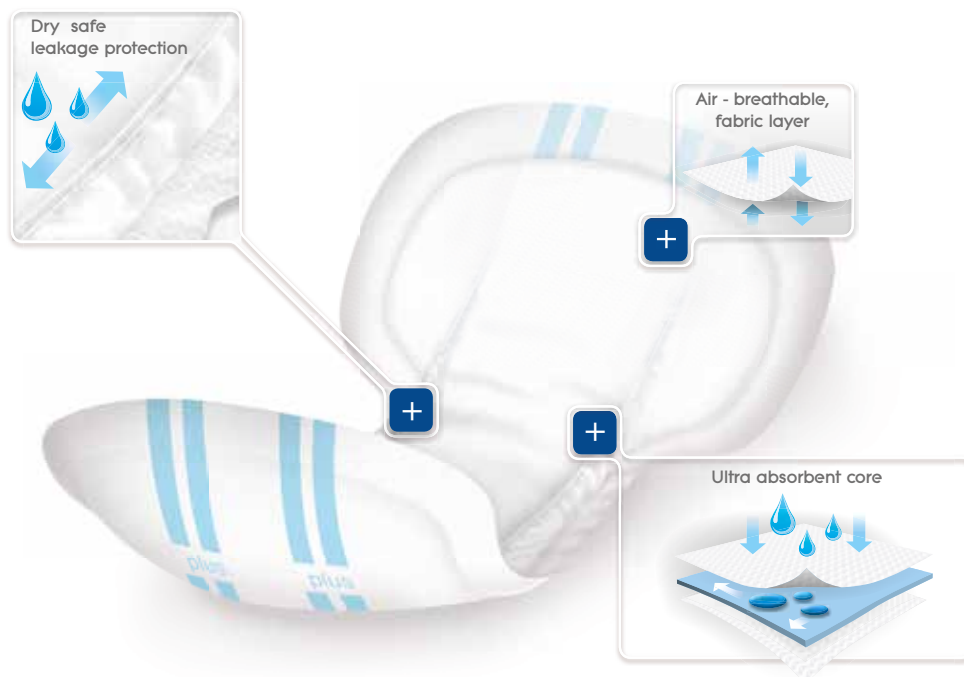

# Large shaped pads




	NPC	MPC	EAN code	Brand name	Colour code	Severity	Size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
Sensitive	CFP1959	5040	4036978900048	forma-care Sensitive form - plus			60 x 32 cm	6	20	1770
	CFP1960	5045	4036978900055	forma-care Sensitive form - super			70 x 35 cm	9	20	2138
	CFP1961	5050	4036978900062	forma-care Sensitive form - extra			70 x 35 cm	12	20	3384
Comfort	CFP1969	7040	4036978900239	forma-care comfort form - plus			60 x 32 cm	6	20	1770
	CFP1970	7040-01	4036978900239	forma-care comfort form - plus S			60 x 32 cm	6	20	1770
	CFP1971	7045	4036978900246	forma-care comfort form - super			70 x 35 cm	9	20	2138
	CFP1972	7050	4036978900253	forma-care comfort form - extra			70 x 35 cm	11	20	3384



# Large shaped pads



## How to fit whilst standing

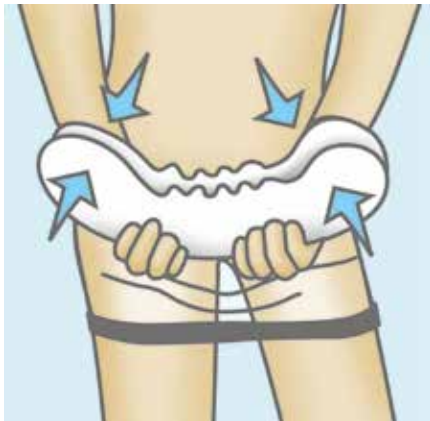

Raise the net fixation pants up to the middle of the thighs. Stretch the pad lengthways and fold along its length to form a small "ship" shape. The wider end fits to the back.




Pull the pad in the "ship" shape from front to rear, above the net fixation pants, through the legs. Unfold the front side of the pad but do not pull the product upwards. Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.




Pull up the net fixation pants. Check the fit of the product, groin and crotch area. The folds of the thighs are positioned outside net the fixation pants.


## How to fit Men's Shaped Pads

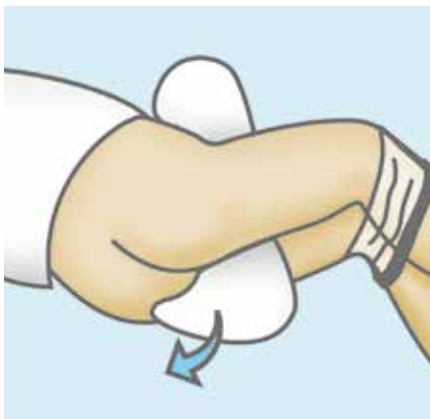

Lie on your side. Pull the fixation pants down below the knees. The wide side is the back. Draw the template in the form of a boat from front to back through the legs.

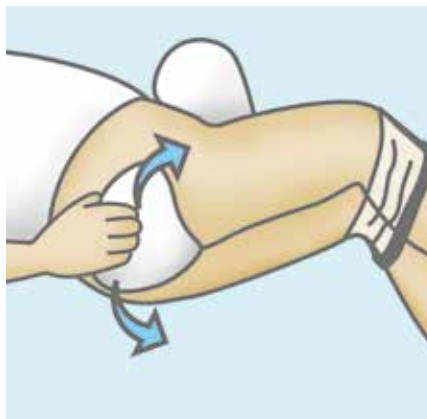

Pull the pad backwards slightly. First unfold the back, then the front. Do not pull the product upwards while doing this. Check that the product is in the centre of your body, by lining up the wetness indicator with the spine.

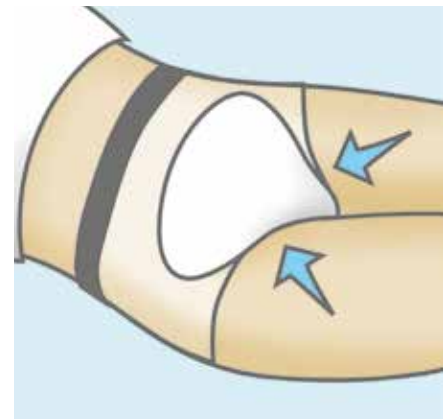

Pull the net fixation pants over the pad. Make sure that the fold of the skin on the thighs are outside the pad and net fixation pants. Check the fit again.

### User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.

## Incontinence severity


(involuntary loss of urine in 4 hours)



At home. In the park. In the restaurant and everywhere else.

**Strong performance for an active life.**

Easy to use, highest comfort and optimum safety provide an „A+“ quality of life.

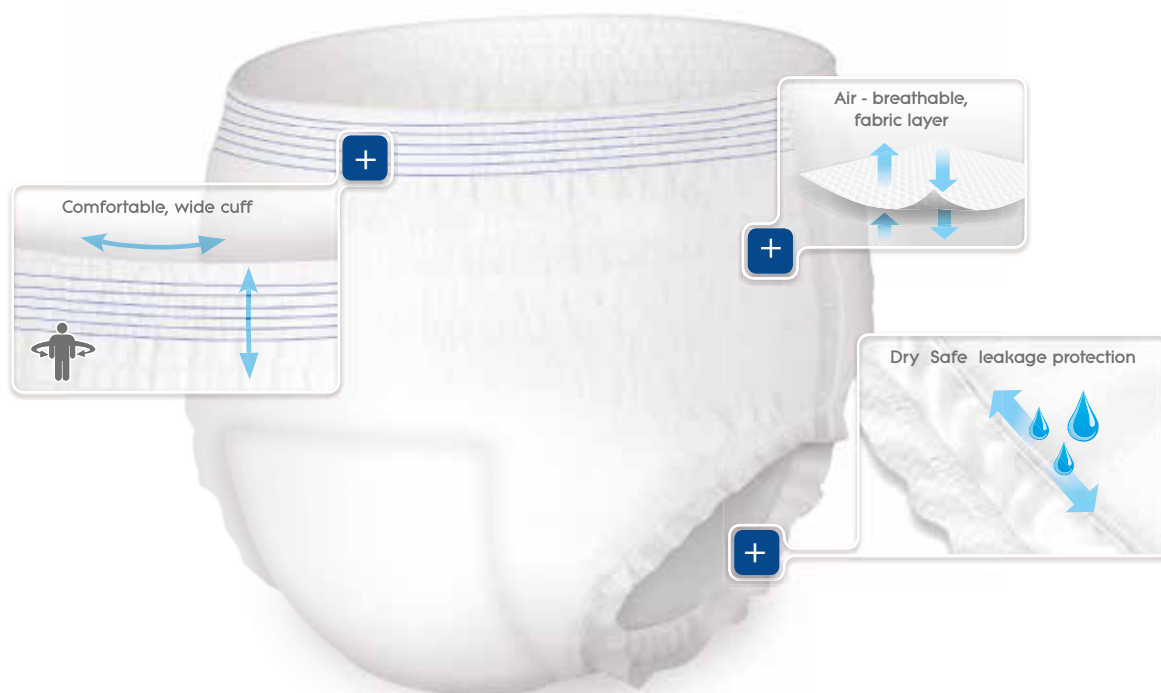

CE





# Pull up pants



PREMIUM

NPC	MPC	EAN code	Brand name	Colour code	Severity	Waist size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
CFP1952	2071	4036978013274	forma-care pants PREMIUM - dry small			50 - 90 cm	5,5	14	1320
CFP1953	2074	4036978013250	forma-care pants PREMIUM - dry medium			75 - 120 cm	5,9	14	1460
CFP1956	2075	4036978013236	forma-care pants PREMIUM - dry medium night			75 - 120 cm	6,3	14	1800
CFP1954	2077	4036978013212	forma-care pants PREMIUM - dry large			100 - 155 cm	6,3	14	1590
CFP1955	2079	4036978013298	forma-care pants PREMIUM - dry x-large			120 - 175 cm	6,7	14	1700
CFP1957	2079-01	4036978013298	forma-care pants PREMIUM - dry x-large			120 - 175 cm	6,7	14	1700



## How to fit whilst standing



Pull the pants up over the legs like under-pants.



Pull up the product so that it fits well into the groin and crotch area. Check the fit once again.



To take off the pants, rip open the side seams and dispose.

### User Instructions:


- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.



# Small shaped pads




## Incontinence severity (involuntary loss of urine in 4 hours)

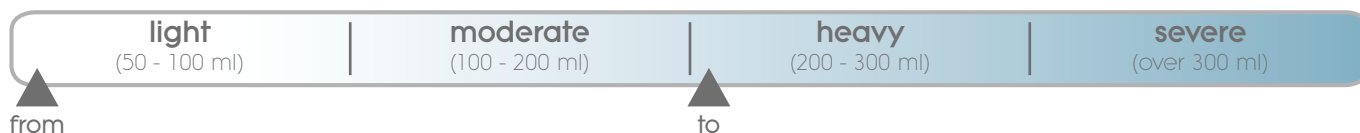

**Thinner and lighter. Hardly noticeable.**

**The pads for the woman and the man of today.**

forma-care shaped pads woman and man active. The little ones with great performance. forma-care products will guide you safely through the day.

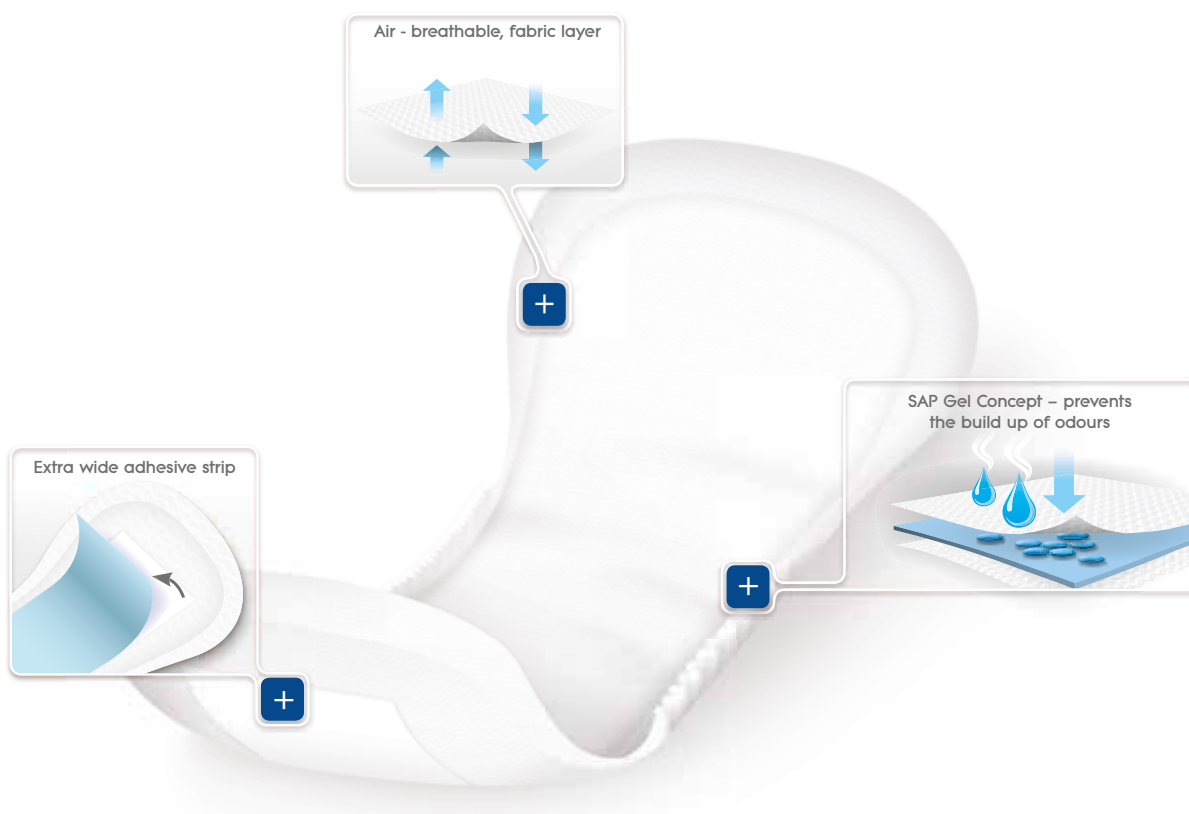

# Datasheet

## Small shaped pads



NPC	MPC	EAN code	Brand name	Colour code	Severity	Size	Weight in kg	Contents/ pack	ISO Absorbency in ml.
CFP1958	20-4031	4036978013311	forma-care woman PREMIUM - mini plus			27,5 x 11 cm	2,3	16	300
CFP1966	4040	4036978900208	forma-care woman - normal			27 x 14 cm	4,3	20	490
CFP1967	4040-01	4036978900208	forma-care woman - normal plus			27 x 14 cm	4,3	20	490
CFP1968	4045	4036978900215	forma-care woman - extra			32 x 15 cm	5	20	815
CFP1962	3010	5703538758430	forma-care man active			41,5 x 15 cm	3,4	20	980
CFP1963	3010-01	5703538758430	forma-care man active			41,5 x 15 cm	3,4	20	980
CFP1964	3010-02	5703538758430	forma-care Man active			41,5 x 15 cm	3,4	20	980
CFP1965	3010-03	5703538758430	forma-care Man active			41,5 x 15 cm	3,4	20	980




# Small shaped pads



## How to fit small shaped pads



Fold the product along the center of the pad into the shape of a "ship", then peel off the protective film from the insert.

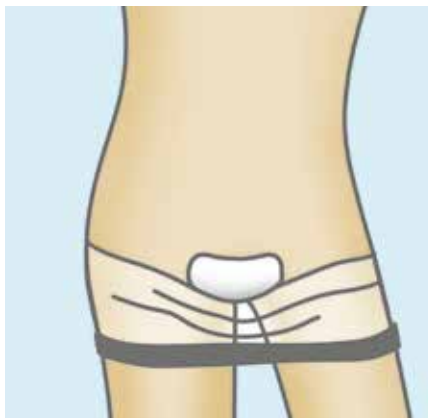


Place the insert with the adhesive side down into the underwear.




Pull up the underwear or net fixation pants to fit.

## How to fit Men's Shaped Pads

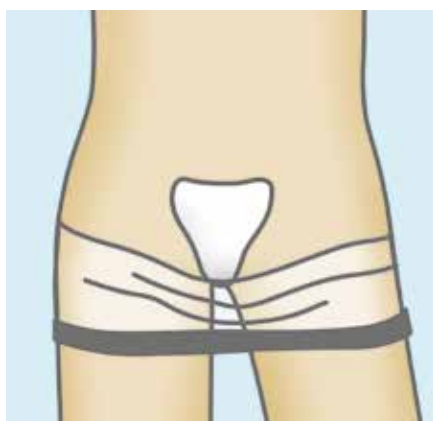

Remove the protective strip so that the adhesive surface is released.

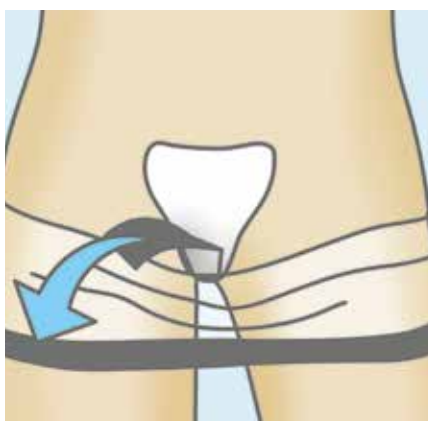

Fold the insert lengthwise to a small ship shape. Place the penis and testicles into the insert.



Pull up the underpants or fixation pants.

### User Instructions:

- Never shake or wring out the products.
- Only fold or stretch the products, if the instructions for folding the „ship“ are given.
- Do not allow contact with pointed objects when using the products – do not puncture.
- Do not work with dirty or greasy hands or when using care products. Make sure that no grease is caught on the outer layer, the adhesive strip or the inside layer of the product.